



MalaDiaKit (MDK)

Adult

Ed. 2010
CSRMDKE

The MDK is a kit that combines communication, diagnosis and treatment of a *P. falciparum* malaria attack. It is intended for use by the local personnel of companies based in malarial zones that wish to play an active role in the health of their employees and the surrounding community as part of their policy of Corporate Social Responsibility .

MDK ADULT

Contents

- 1 electronic thermometer
- 1 rapid diagnostic test for malaria (*P. falciparum*)
- 1 box of a complete treatment for symptoms of malaria
- 1 box of a complete treatment for an attack of malaria
- 1 customizable and descriptive jacket

Use of rapid diagnostic test

1. Hold a lancet by its sides
2. Turn the movable blue cap to free it
3. Carefully disinfect the tip of a finger
4. Apply the exposed needle to the finger
5. Press trigger (a painless prick)
6. Press finger to produce a drop of blood
7. Apply the blotting test end of strip to the drop of blood
8. Insert strip into tube
9. After 5-15 minutes read the result



Instructions for treatment of symptoms

1 to 2 pills every 4 hours. Do not exceed 6 pills per day: read drugs instructions carefully before using.

Instructions for treatment of malaria attack

2 pills per day once daily, during 3 days: read drugs instructions carefully before using.

The rapid diagnosis and home treatment afforded by the MDK kits are recommended by the WHO as part of its commitment to fight malaria. The MalaDiaKit is also a powerful communications tool for businesses, both internally and externally



Solutions For Your Good Health