



Malaria Survival Kit

Multiple Species

This kit is designed to diagnose and rapidly treat the 4 different strains of malaria to avoid the risk of death. It should always be kept within reach when one has been exposed to malaria. It will be especially useful when returning from a malarial zone, when a traveler is in a country where malaria is unknown.



DIAGNOSTIC

Contents

- 1 forehead thermometer
- 3 plastic lancets
- 3 white packets
- 3 rapid diagnostic tests for malaria falciparum and other species:
 - 3 Immunoquick malaria +4 strips
 - 3 tubes of reagent
- 3 disinfectant wipes

Use of Rapid Diagnostic Test

1. Hold a lancet by its sides
2. Turn the movable blue cap to free it
3. Carefully disinfect the tip of a finger
4. Apply the exposed needle to the finger
5. Press trigger (a painless prick)
6. Press finger to produce a drop of blood
7. Apply the blotting test end of strip to the drop of blood
8. Insert strip into tube
9. After 10 minutes read the result

TREATMENT

Contents

- 1 box of a complete treatment for an attack of malaria (24 tablets).

Use of medicine

1. Immediately take 4 tablets of this medicine with food or milk.
2. Take 4 more tablets 8 hours later
3. Continue taking 4 tablets mornings and evenings to complete the entire curative treatment of 24 tablets. This takes 3 days.

Once you have started the treatment, it must be completed, even if you begin to feel better.



Wherever possible, use your Malaria Survival Kit Multiple Species in consultation with a doctor. If you are unable to see one rapidly (within 6 hours following the first sign of symptoms), you can perform the malaria diagnostic test yourself, preferably with the assistance of a person accompanying you.



Solutions For Your Good Health

Ed. 2010
HKP050E